


Application of Ambu® BlueSensor™ Electrodes

BlueSensor by Ambu


1

1. Clean with soap/water; avoid alcohol as it dehydrates skin.
2. Dry thoroughly.
3. Clip/shave hair only if needed.
4. Gently abrade to reduce impedance.

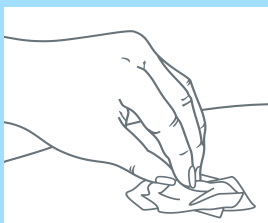


2

Peel electrode from the plastic liner and place in the appropriate recycling container, where available.

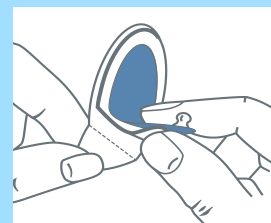


3

Place the electrode on the skin by gently pressing around the adhesive edge with a circular motion.

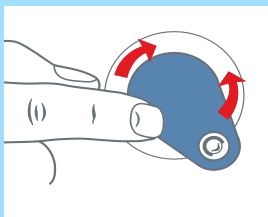


4

When using electrodes with wet gel, avoid pressing down on the center of the electrode.

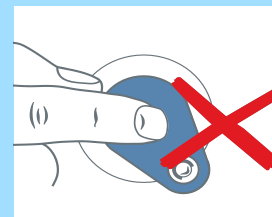


5

Connect the cable

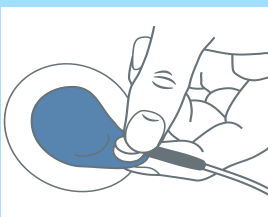

6

Electrodes with offset connectors should be positioned facing in the same direction as the cable.

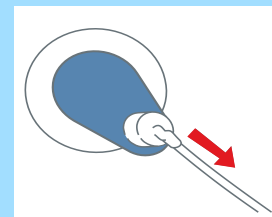

SCAN FOR
INSTRUCTION
VIDEO

Questions? Contact your Ambu representative at
1300 233 118 or visit www.ambuaustralia.com.au

Ambu

BlueSensor

Ambu Australia Pty Ltd | Ambu New Zealand Pty Ltd
Unit 5, 2 Daydream Street, Warriewood, NSW 2102
Tel (AUS): 1300 233 118 | Tel (NZ): 0800 804 079
Email: aussales@ambu.com Web: www.ambuaustralia.com.au

Always read the Instructions for Use. These products
are only available for sale to healthcare professionals.

AA26 1304 V01 04/2026 Ambu Australia
Technical data may be modified without further notice.