DATASHEET

Ambu AuraFlex is a disposable, flexible laryngeal mask which is specially designed for ENT, opthalmic, dental and other head and neck surgeries.

AuraFlex has a long, wire-reinforced, and flexible airway tube that allows positioning away from the surgical field without loss of seal. The kink-free wire-reinforced airway tube prevents airway tube occlusion.

The integrated pilot tube makes handling and insertion easy.

AuraFlex comes in a full range from size 2 to size 6.

Ambu[®] AuraFlex



KEY BENEFITS

High flexibility of the reinforced airway tube enables positioning away from the surgical field

Kink-free wire-reinforced airway tube prevents airway tube occlusion

Full range from size 2 through size 6

Soft cuff capable of establishing a seal with low cuff pressure

Cuff design prevents backfolding

Depth marks for visual control of correct positioning of the mask

Packaged sterile and ready for use

Colour-coded packaging with directions for use



Ĕ
ė
₽.
0
č
ē
÷
.5
ŧ
5
¥
ó
ě
difie
۰
Ε
Ф
٠
ě
Ε
ū
ā
9
g
Ē
운
ě
Ś
>
``
호
Ε,
ч
ò
Ξ
022/
022
2
ő
0
_
2
藍
Ö
오
œί
_

	Paediatric		Adult			
Mask size	#2	#21/2	#3	#4	#5	#6
Packaging color code						
Patient weight	10 - 20kg	20 - 30kg	30 - 50kg	50 - 70kg	70 - 100kg	>100kg
Maximum intracuff volume	10ml	14ml	20ml	30ml	40ml	50ml
Maximum intracuff pressure	60 cmH₂O					
Maximum instrument size*	5.5mm	5.5mm	7.0mm	7.0mm	8.0mm	8.0mm
Airway connector	15mm male (ISO 5356-1)					
Inflation Valve Luer cone compatibility	Luer cone compatible with ISO 594-1 and ISO 80369-7 compliant equipment					
Appropriate storage condition	10 °C (50 °F) to 25 °C (77 °F)					
Approximate mask weight	22g	25g	34g	39g	49g	54g
Internal volume of ventilatory pathway	$8.0 \pm 0.3 ml$	$8.6 \pm 0.8 \text{ml}$	13.2 ± 0.8 ml	13.6 ± 1.2ml	17.6 ± 1.5ml	18.5 ± 0.8 ml
Pressure drop as determined according to ISO 11712 annex C	0.8cmH₂O at 30 l/min	0.7cmH₂O at 30 l/min	0.8cmH₂O at 60 l/min	0.7cmH₂O at 60 l/min	0.6cmH₂O at 60 l/min	0.6cmH₂O at 60 l/min
Min. interdental gap	17mm	20mm	21mm	24mm	27mm	29mm
Nominal length of the Internal ventilatory pathway	21.2 ± 1.3cm	21.2 ± 1.3cm	23.7 ± 1.4cm	23.7 ± 1.4 cm	26.2 ± 1.6cm	26.2 ± 1.6cm

^{*}The maximum instrument size is intended as a guide for selecting the appropriate diameter of a device to be passed through the patient tube of AuraFlex.

MATERIALS

Product	Phthalate-free	Material		
Patient tube	Yes	PVC/Stainless Steel		
Cuff	Yes	PVC		
Pilot tube	Yes	PVC		
Check valve	Yes	PP, PC, Green Silicone Rubber		
Connector	Yes	PCTG		
The Ambu AuraFle	ex is not made with natural	rubber latex		
Packaging	Quantity			
Pouch	1	Tyvek®/HDPE		
Вох	10	Cardboard		



AuraFlex family - 6 sizes: 2 through 6

ORDERING INFORMATION

Product Code	Size	Туре
327 200 000	2	Disposable Flexible Laryngeal Mask
327 250 000	21/2	Disposable Flexible Laryngeal Mask
327 300 000	3	Disposable Flexible Laryngeal Mask
327 400 000	4	Disposable Flexible Laryngeal Mask
327 500 000	5	Disposable Flexible Laryngeal Mask
327 600 000	6	Disposable Flexible Laryngeal Mask



AuraFlex cuff



Ambu Australia Pty Ltd Unit 5, 2 Daydream Street Warriewood NSW 2102 Tel: 1300 233 118 aussales@ambu.com www.ambuaustralia.com.au

